

Successful Removal of Bilateral Embolization Using the Pounce™ Thrombectomy System



Bruce H. Gray, DO, MSVM

Professor of Surgery/Vascular Medicine
University of South Carolina School of Medicine
Greenville, South Carolina



Scan to read this full case report or
visit pouncesystem.com/casereports

Patient Presentation

- 44-year-old female mail carrier presented with a 4-week history of claudication.
- No risk factors for atherosclerosis, normal sinus rhythm, and no history of medical illness.
- Initial arteriogram revealed bilateral embolization that causes an occlusion of the descending branch of the right profunda femoral artery, right popliteal artery occlusion, and left tibioperoneal trunk occlusion.

Procedure Overview

- Bilateral CFA access enabled the Pounce™ Thrombectomy System to remove the emboli and reestablish normalized flow.
- No use of thrombolysis or surgical intervention.
- Multiple vessels and branches of the arterial tree were able to be reopened during the same session.

PRE-TREATMENT IMAGING

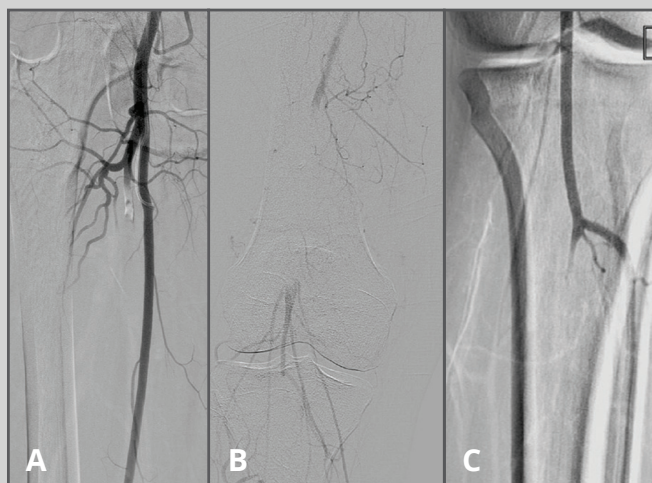


Figure A: Baseline Arteriogram of Right Profunda Femoral Artery Occlusion

Figure B: Right Popliteal Artery Occlusion

Figure C: Left Tibioperoneal Trunk Occlusion

FINAL IMAGING

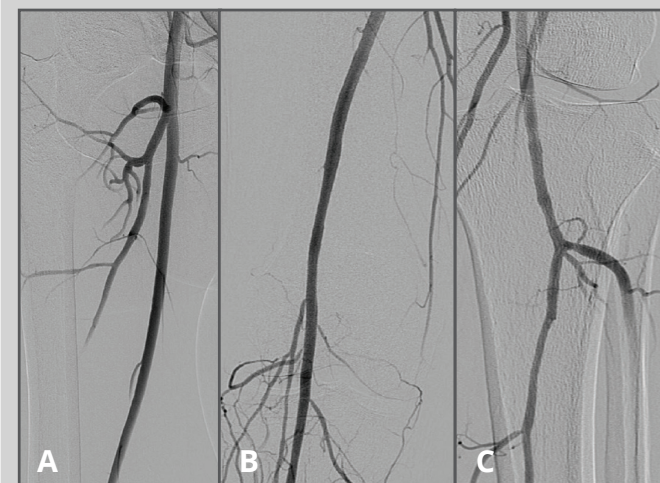



Figure A: Right Profunda Femoral Artery after Pounce™ Thrombectomy System Passes

Figure B: Right Popliteal Artery after Pounce™ System Passes

Figure C: Left Tibioperoneal Trunk after Pounce™ System Passes

CLOT REMOVED

Clot Removed from the Right Profunda Artery

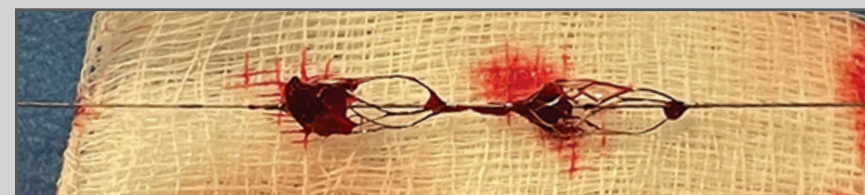

Conclusion: The Pounce system was able to normalize flow without an incision or exposing the patient to thrombolytic therapy.



SURMODICS, POUNCE, and SURMODICS and POUNCE logos are trademarks of Surmodics, Inc. and/or its affiliates. Third-party trademarks are the property of their respective owners.

All images used with permission of author.
©2023 Surmodics, Inc. All rights reserved. SRDX-PTS-803 A 5/23

Caution: Federal (US) law restricts this device to sale by or on the order of a physician. Please refer to Instructions for Use for indications, contraindications, warnings, and precautions.

www.pouncesystem.com